




2 A Teschio e denti, 1489.

RL 19058 v (B. 41 r; K/P 42 v).

Il vacuo della cassa dell'occhio, e 't vacuo dell'osso sostenitore della guancia, e quello del naso e della bocca sono d'eguale profondità e terminano sotto il senso comune [sede centrale delle sensazioni] per linea perpendicolare. E ciascuna d'esse vacuità ha tanto di profondità quant'è la terza parte del volto dell'omo, cioè dal mento a capegli.

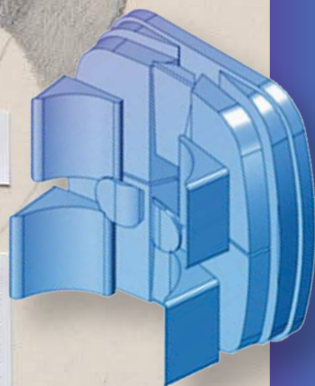

**A NEW SOLUTION**  
THE TRUNK TOTAL SUPPORT

# A NEW SOLUTION

## TTS TRUNK TOTAL SUPPORT

### Positioning kit for the trunk with modular inserts



## Why Conco ?

- FINALLY THE POSITIONING BACKREST WITHOUT MOUNTING HARDWARE AND ALUMINIUM SHELL
- LIGHTWEIGHT, COMFORTABLE, SAFE
- FLEXIBLE TO MAXIMISE MOVEMENTS
- PERFECT FIT, SUPPORT AND CONTROL TO THE TRUNK INDIVIDUALLY AND IMMEDIATELY ADAPTABLE WITHOUT TOOLS



STRUCTURAL KIT,  
MODULAR AND MODIFIABLE  
WITH 16 INSERTS



POLYURETHANE ELASTIC FOAM  
OR MEMORY FOAM



AVAILABLE WITH OR WITHOUT  
MOUNTING MULTI-TENSION SYSTEM  
\* IF PROVIDED, THE BACKREST IS DEPTH,  
HEIGHT AND INCLINATION ADJUSTABLE

## CLINICAL INFORMATION/DESTINATION OF USE

The trunk positioning system Inserto Back Conco:

- Is recommended preferably, but not exhaustively for users with HIGH risk of pressure injuries.
- Is suggested also in combination with orthosis for the rachis.
- Embraces the rachis gibbities and protrusions to distribute the load.

**TTS** EMBRACING SHAPES

## TECHNICAL FEATURES

- Structural kit composed by:
  - Flat base of construction and 16 polyethylene closed celled inserts, modular and modifiable, useful to individually support or correct the posture of the trunk during the life time use.
- Padding available in a single layer of polyurethane elastic foam or memory foam.
- Microclimatic ventilated cover, fireproof and antibacterial. It can be washed in tepid water with a gentle detergent. Do not tumble dry. The fabric doesn't need the use of disinfectants.
- The optional mounting multi-tension strap system is available in two versions:
  - Single straps (which number depends on the backrest height).
  - The combination of a central double strap and single straps positioned on the upper and/or lower part of the backrest.

\* Follow the information contained in the product technical sheet of the instruction of use manual to install the backrest on the wheelchair.

## WARNINGS

The trunk positioning system Inserto Back Conco, in its design integrity and numerical and dimensional completeness of the supplied components (structural kit, padding, cover) can be easily adapted to sizes/morphology/deformities of the user. This kind of operations make the commissioning referable to a serial device.

Alternatively

The commissioning of the Positioning System Inserto Back Conco, deprived or modified, even partially, of its design integrity of numerical and dimensional completeness of the supplied components, built as per written prescription of a professional operator in function of the anatomy/morphology/deformities of the user, through the measurement detection and direct trials, can be referable to a custom device.

**SPARE PARTS** ■ Padding ■ Cover ■ Multi-tension strap system with single straps ■ Multi-tension strap system with central double strap and single straps



# CONCO



## Inserto Conco

16 MODULAR AND CONFIGURABLE INSERTS



MICROCLIMATIC  
VENTILATED  
COVER


° Each postural unit is ordered based upon the size of mobility/wheelchair device

**INSERTO Back CONCO** (Structural kit with padding, cover, multi-tension strap system with H 10 cm double strap and H 5 cm single straps)

Model (cm)	2415	2421	2815	2821	3215	3221	3227	3618	3624	3630	4018	4024	4030
Effective shell width (cm)	19	19	23	23	27	27	27	31	31	31	35	35	35
Effective shell height (cm)	15	21	15	21	15	21	27	18	24	30	18	24	30
*Effective wheelchair external tubes width	24-26	24-26	28-30	28-30	32-34	32-34	32-34	36-38	36-38	36-38	40-42	40-42	40-42
H 5 cm single straps	3 straps	4 straps	3 straps	4 straps	3 straps	4 straps	5 straps	3 straps	4 straps	5 straps	3 straps	4 straps	5 straps
H 10 cm double central strap and H 5 cm single straps	✗	✓	✗	✓	✗	✓	✓	✗	✓	✓	✗	✓	✓

\* indicative values

conversion table	inches	6	7	8	9,5	10,5	11	12	12,5	14	16
	cm	15	18	21	24	27	28	30	32	36	40

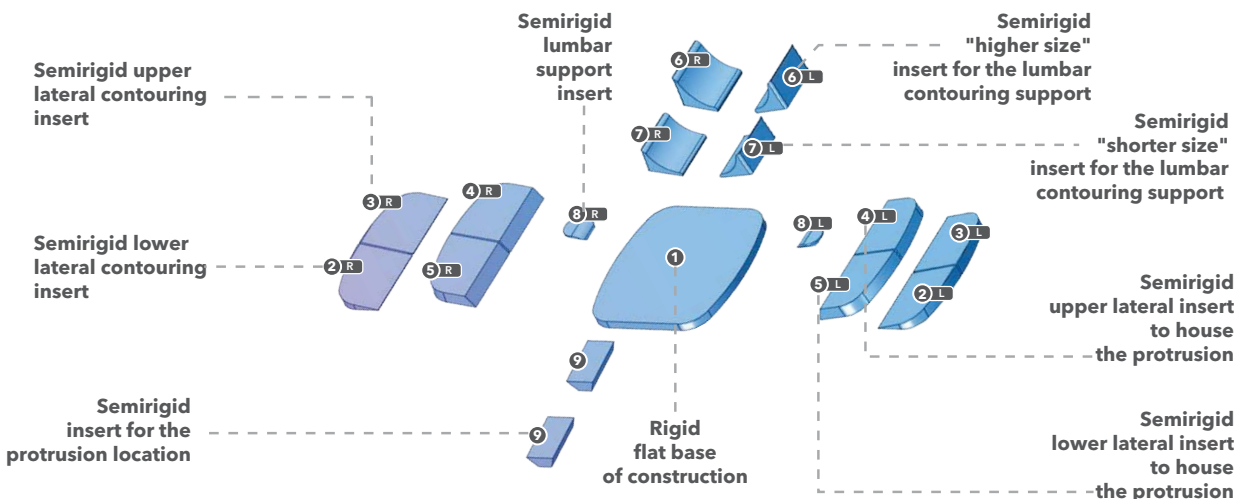

For partially sighted users this document can be visualised in PDF format on the web site [www.versainserto.com](http://www.versainserto.com) with the possibility to enlarge the text.

# CONCO inserto<sup>®</sup> BACK

MANUFACTURED BY




Pro Medicare srl

Via Montagna Z.I. - Lotto 41  
72023 Mesagne (BR) - ITALY  
Tel. +(39) 0831 777840  
Fax. +(39) 0831 730739  
Email: sales@promedicare.it  
www.versainserto.com  
www.promedicare.eu

CERTIFIED COMPANY WITH QUALITY  
MANAGEMENT SYSTEM ISO 9001 / EN ISO 13485  
N° 279412 / N° 279411



DISTRIBUTED BY

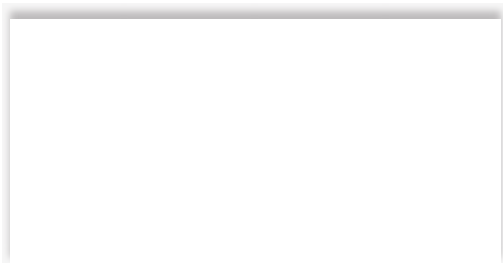

CE 2 YEAR WARRANTY

[www.versainserto.com](http://www.versainserto.com)

The manufacturer reserves the right to make changes or replacements without notice.  
The contents, the form, the creation, the design, the image and the structure of this catalog are the property of Pro Medicare srl.  
Any improper and unauthorised use is strictly forbidden. Any imitation and/or explicit or not explicit copy can be legally persecuted.  
Cover works by artist Giuseppe Ciraci - [www.giuseppeciraci.com](http://www.giuseppeciraci.com) - Any reproduction is explicitly forbidden.