

Thinking **MODO**
inserto[®]
SPINE

SPINE

SEAT

2 A Teschio e denti, 1489.

R.I. 19058 v (B. 41 r; K/P 42 v).

Il vacuo della cassa dell'occhio, e 'i vacuo dell'osso sostenitore della guancia, e quello del naso e della bocca sono d'eguale profondità e terminano sotto il senso comune [sede centrale delle sensazioni] per linea perpendicolare. E ciascuna d'esse vacuità ha tanto di profondità quant'è la terza parte del volto dell'omo, cioè dal mento a capegli.



A NEW SOLUTION
INDIVIDUALLY CUSTOMISABLE

A NEW SOLUTION INDIVIDUALLY CUSTOMISABLE Construction kit of the individually customisable seat Perfect reproduction of anatomical shapes



Why Modo ?

- DEVELOPED FOR SPINAL CORD INJURIES USERS
- OPTIMAL LOADS DISTRIBUTION
- ULTRA LIGHTWEIGHT
- VERY COMFORTABLE



MODULAR AND CONFIGURABLE
PRE-SHAPED STRUCTURAL KIT



PADDING SYSTEM
Free Shape PROTECTIVE
AND SHOCK ABSORBING



COMFORTABLE AND SAFE
COVER WITH POCKET

CLINICAL INFORMATION / DESTINATION OF USE

The pelvis positioning system Inserto Spine Modo is recommended preferably, but not exhaustively, for users with spinal cord injuries, in particular:

- Those who need a low/mild positioning support;
- Those who lead an active life and need a cushion easy to transport and transfer from;
- Those who have mild/medium risk to develop pressure sores and need a coccygeal and trochanteric discharge;
- Those who have leg length discrepancies, pelvic rotation and obliquity, posterior and anterior pelvis;
- Those who have lack of gluteal mass.

The pelvis positioning system has to be used with any supporting/wheelchair base properly fixed through suitable gripping systems.

PTS Pelvic Total Support

TECHNICAL FEATURES

Each positioning system Inserto MODO includes:

- A structural kit composed by a pre-shaped contentive base and 8 polyethylene closed cells insert, modifiable and configurable, useful to individually support or correct the posture of the pelvis during the life time use;
- Kit of padding, washable with a gentle detergent, which is composed by different layers:
 - A superior layer of memory foam for the highest protection from risk of decubitus issues and an inferior layer of visco-elastic foam with low spring back action, only in the posterior part to enhance maximum comfort and adaptation to body shapes;
 - A central pad of polyurethane open celled foam with a pad of self-modelling polymeric foam and two pads of a self-modelling polymeric foam for levelling or increasing ischium protection;
- New 3D cover, air-exchange, incontinent, not flammable, antibacterial. It can be washed in tepid water with a gentle detergent. Do not tumble dry. The fabric doesn't require the use of disinfectants.

SPARE PARTS

- Pre-shaped base
- Cover
- Kit of padding

WARNINGS

The Positioning System Inserto Seat Modo, in its design integrity and numerical and dimensional completeness of the supplied components (structural kit, padding kit, cover) can be easily adapted to sizes/morphology/deformities of the user. This kind of operations make the commissioning referable to a serial device.

Alternatively

The commissioning of the Positioning System Inserto Seat Modo, deprived or modified, even partially, of its design integrity of numerical and dimensional completeness of the supplied components, built as per written prescription of a professional operator in function of the anatomy/morphology/deformities of the user, through the measurement detection and direct trials, can be referable to a custom device.

For partially sighted users this document can be visualised in PDF format (with the possibility to enlarge the text) on the web site www.versainserto.com





Inserto Modo

8 MODULAR AND CONFIGURABLE INSERTS

"NEW 3D" COVER
STRECH, INCONTINENT, AIR-EXCHANGE.

° Each postural kit is ordered based upon the size of mobility/wheelchair device.


INSERTO MODO SIZES TABLE													SIZES (cm.)		
MODEL	3638	3642	3840	3845	4040	4045	4242	4245	4250	4545	4550	4848	4850	5050	
Effective Width (cm)	36	36	38	38	40	40	42	42	42	45	45	48	48	50	
Min. width achieved with removal of material (cm)	34	34	36	36	38	38	40	40	40	43	43	46	46	48	
Effective Length (cm)	38	42	40	45	40	45	42	45	50	45	50	48	50	50	
Max length achieved with adaptation (cm)	40,5	44,5	42,5	47,5	42,5	47,5	44,5	47,5	52,5	47,5	52,5	50,5	52,5	52,5	
Min. length achieved with removal of material and removal of inserts (cm)*	36	40	38	43	38	43	40	43	48	43	48	46	48	48	
Min. and max. range of anterior height pre-shaped construction + leg housing inserts 1/2/3 (cm)**	4,5 / 5,5		** height referred to models 3638 and 5050												
Min. and max. range of posterior height of pre-shaped base construction (cm)**	8 / 9		** height referred to models 3638 and 5050												
Weight of positioning system (min./max): 1 Kg / 2,5 kg															

* possible and further reductions due to removal of material

° Max. Load (referred to MODEL 5050): 135 Kg.

CONVERSION TABLE


INCHES	10	12	12,5	13	13,5	14	15	15,5	16	16	16,5	16,5	17	17,5	17,5	17,5	18	18,5	18,5	19	19,5	19,5	20	20	22
CM.	25	30	32	33	34	36	38	39	40	40,5	42	42,5	43	44	44,5	45	46	47	47,5	48	50	50,5	52	52,5	55

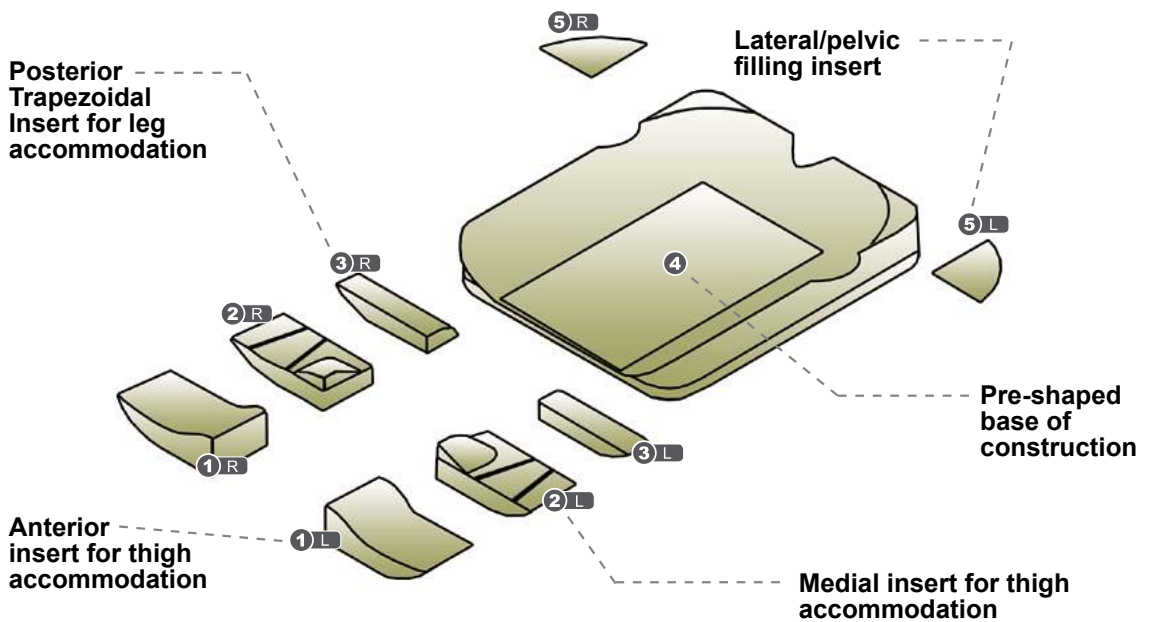




O D O


Customisations of Modo

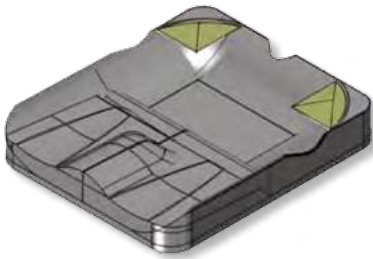


FILLING THE GLUTEAL MASS AND LOADS CUSTOMISATION

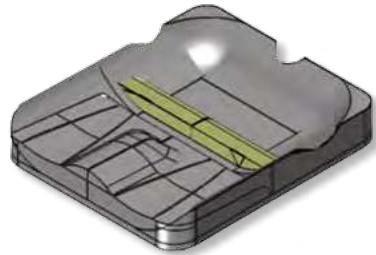


ADJUSTMENT OF THE PRE-ISCHIUM BLOCK

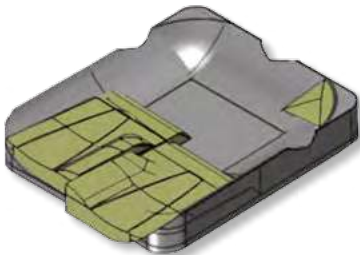

ASYMMETRY DUE TO PELVIC ROTATION

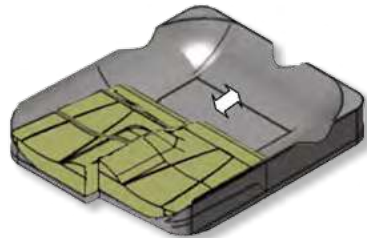

CUSTOMISATION OF THE DEPTH

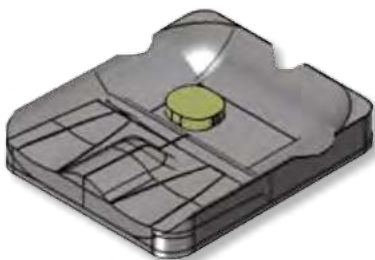

PELVIC OBLIQUITY MANAGEMENT ISCHIAL UNLOAD

INDIVIDUALLY CUSTOMISABLE

Construction kit of the individually customisable seat



AN INNOVATIVE SOLUTION

A range developed to answer the individual postural need (as per prescription) and numerous pathologies throughout the whole life cycle of use.

In order to maintain its features it is necessary to periodically check the conservation and wear of all the parts composing the device as well as all hook and loops stripes.

1 PREVENTS THE PRESSURE OVERLOAD

Bigger is the material that supports the body load, less is the stress of the deep tissues and of the cushion that supports the load.

2 ADAPTABILITY TO THE INDIVIDUAL MORFOLOGY AND ANATOMY

In order to realise the customised postural solution.

3 PTS PELVIC TOTAL SUPPORT

Adherence to the anatomy of the pelvis and of the related body segments. It enhances the active response of the musculo-skeletal system. It supports the sacral orientation and guarantees the containment of relationship system legs/pelvis/rachis.

4 FREE SHAPES PADDING

The innovative modular padding ensures maximum comfort and reliable protection.






MANUFACTURED BY:



Via Montagna Z.I. - Lotto 41
72023 Mesagne (BR) - ITALY
Tel. +(39) 0831 777840
Fax. +(39) 0831 730739
Email: sales@promedicare.it
www.versainserto.com
www.promedicare.eu

CERTIFIED COMPANY WITH QUALITY
MANAGEMENT SYSTEM ISO 9001 / EN ISO 13485
N° 279412 / N° 279411

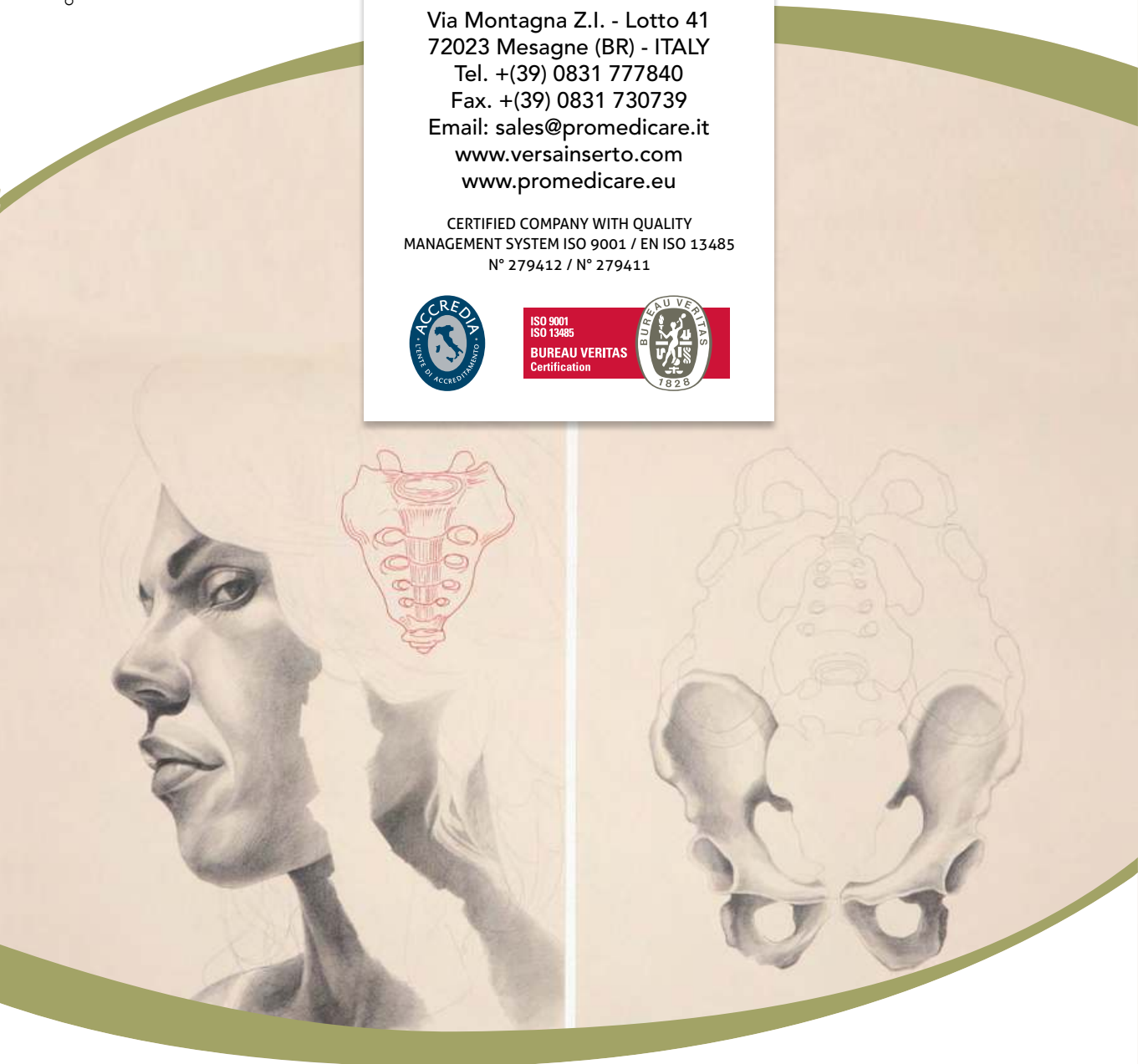

DISTRIBUTED BY:



CE 2 YEAR WARRANTY

www.versainserto.com